


Sheep Artificial Insemination Fact Sheet 6: Assembly of a Sheep Artificial Vagina

You will need:

- Valve for filling the AV with water (doesn't have to be as large as pictured, just something that will allow you to fill the AV with water)
- A short piece of PVC pipe
- Rubber tubing
- A rubber funnel
- 10 cc centrifuge tube
- 50 cc centrifuge tube with a hole in the lid
- Large hose clamp

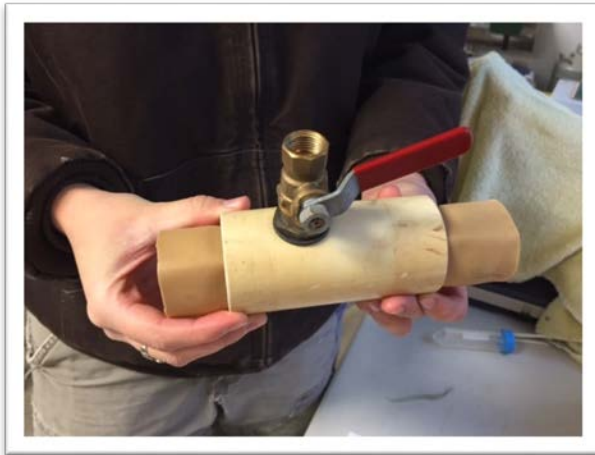

Place rubber tubing inside PVC and fold ends over the pipe





*Place funnel on end of PVC pipe
and secure with the large hose
clamp*



Place the 5cc centrifuge tube into the hole in the lid of the 10cc centrifuge tube



*Fit centrifuge tubes
onto the end of the AV
to catch semen.*

The AV will be filled with warm water and lubricated prior to use. The 10cc centrifuge tube will also be filled with warm water to keep the semen warm during collection. (See Factsheet on Collection with an AV)

